Comparison of **Disposable vs. Reusable** Robotic Bronchoscopes

Krish Bhadra, MD



Reusable bronchoscopes have been a staple of first-generation robotic bronchoscopy systems

Both the Monarch[™] platform, launched in 2018, as well as the Ion[™] Endoluminal platform, launched in 2019, offer reusable bronchoscopes, while the second-generation platform from Noah Medical – the Galaxy System[™] – offers a disposable bronchoscope.¹

While the Ion system uses fiberoptic bend sensors to maintain orientation in space, the Monarch's and Galaxy's scopes use Electro-magnetic navigation (EMN) to maintain spatial orientation. The Galaxy system also includes integrated digital tomosynthesis capability.¹

Table showing key attributes of the different scopes¹

	Monarch Robotic Bronchoscopy System	Ion Endoluminal System	The Galaxy System
Navigation Technology	EMN	Shape Sensing	EMN with digital tomosynthesis (TiLT+ Technology")
Outer Diameter	Outer Sheath: 6mm Inner Scope: 4.2mm	3.5mm	4.0mm
Working Channel Diameter	2.1mm	2mm	2.1mm
Vision during Bx	Yes	No	Yes
Scope Reprocessing	Yes	Yes	Not needed (disposable)

It is important to note that, unlike the Monarch and Galaxy's scopes which have integrated vision capability, the lon's scope doesn't. Instead, vision is achieved through using a vision probe inserted through the scope, but which must be removed at the time of biopsy. The vision probe would incur an additional cost.

Several organizations (such as AORN, SGNA, ANSI/AAMI etc,.)
provide reprocessing standards and guidelines.

Depending on the organization, the number of steps
involved in reprocessing could vary

up to 34 steps.²



Single-use disposable bronchoscopes offer several advantages over reusable bronchoscopes

1) Hidden costs of reprocessing re-usable bronchoscopes

- Reprocessing one flexible endoscope requires approximately 76 minutes of hands-on staff time.²
- The cost of reprocessing one endoscope ranges from \$114.07 to \$280.71.2

Disposable bronchoscopes have a straightforward purchasing cost and **require no reprocessing**, thus improving efficiency.

Reusable bronchoscope reprocessing requires

~**76** minutes

of hands-on time²

Disposable bronchoscopes require no reprocessing

Average cost of treating an Acute Respiratory Infection in North America is

~\$29,000°

Disposable scopes are sterile, reducing the risk of cross-contamination

2) Cross-contamination risk and associated treatment costs

- Reusable bronchoscopes have a reported incidence of cross-contamination or infection of 2.8%.³
- Microbial growth and residual contamination was seen in 58% and 100% of inspected scopes respectively, after manual cleaning.⁴
- Average cost of treating an Acute Respiratory Infection in North America is ~\$29,000⁵
- Pathogen transmission can occur even when bronchoscope cleaning and disinfection practices align with current guidelines.⁴

Disposable scopes are provided sterile and require no sterilization or disinfection before use, thus **mitigating nosocomial cross-contamination risks**.

3) High negative environmental impact

A comparison of the production and disposal of single-use bronchoscopes with the re-processing of reusable bronchoscopes (each time) shows the following results:


- Re-usable bronchoscopes have about 80% higher CO2-equivalent emissions than single-use bronchoscopes.⁶
- Energy requirement for re-usable bronchoscopes is **90% higher** than for single-use bronchoscopes.⁶

It is important to note that the exclusion of the manufacturing and disposal of the RBs suggests that the assessment of this option is fairly conservative.

Re-usable bronchoscopes energy requirements are

90% higher


than for single-use bronchoscopes⁶

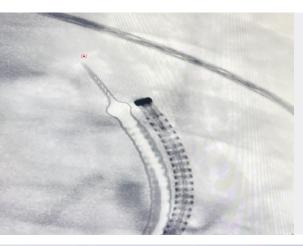

4) Impact on Scope performance

Flexible endoscopes are particularly difficult to disinfect and easy to damage because of their intricate design and delicate materials.⁷

- Re-usable bronchoscopes were seen to have visible defects after manual cleaning in 100% of inspected scopes.⁴
- Scope drift a situation, as shown in the image to the left, where the stability and targeting of the scope is compromised, when a tool is inserted may occur due to scope fatigue from frequent reprocessing.

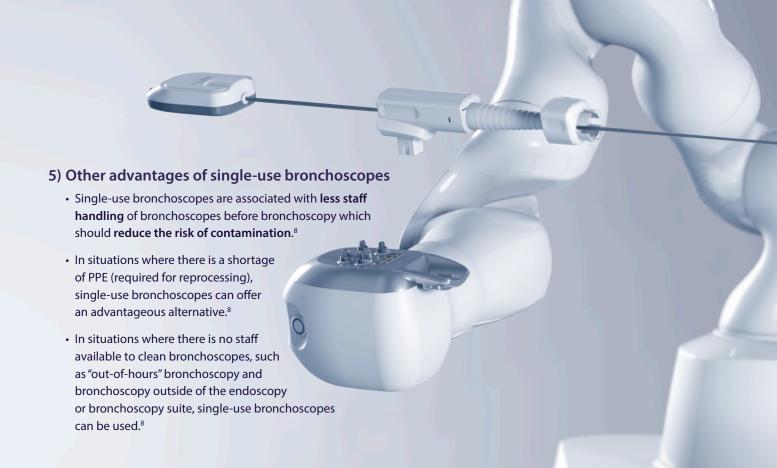

Summary

When assessing the cost of using either single-use scopes or re-usable scopes, it is important to factor in the various hidden costs, higher risk of cross-contamination, and the higher negative environmental impact associated with re-usable scopes.

A disposable scope improves efficiency, reduces the burden of hospital staff and helps to de-risk the procedure from cross contamination, infection and product damage caused by reprocessing.

The FDA also recommends that Disposable bronchoscopes should be used where there is increased risk of spreading infection or when there is no support for immediate reprocessing of the bronchoscope.⁹

Bibliography

- Robotic Bronchoscopy: Review of Three Systems Maxwell J. Diddams, and Hans J. Lee PMID: 36836710
- A glimpse at the true cost of reprocessing endoscopes: Results of a pilot project. In International Journal of Healthcare Central Service Material Management

Cori L. Ofstead, MSPH; Mariah R. Quick, MPH; John E. Eiland, MS, RN; Steven J. Adams, RN, CRCST

- 3) A systematic review and cost effectiveness analysis of reusable vs. single-use flexible bronchoscopes
 - J M Mouritsen, L Ehlers, J Kovaleva, I Ahmad, K El-Boghdadly
- 4) Effectiveness of Reprocessing for Flexible Bronchoscopes and Endobronchial Ultrasound Bronchoscopes Cori L Ofstead, Mariah R Quick, Harry P Wetzler, John E Eiland, Otis L Heymann, David A Sonetti, J Scott Ferguson
- 5) Costs of management of acute respiratory infections in older adults: A systematic review and meta-analysis Shanshan Zhang, Pia Wahi-Singh, Bhanu Wahi-Singh, Alison Chisholm, Polly Keeling, Harish Nair, and RESCEU Investigators PMID: 36342670

- Comparative Study on Environmental Impacts of Reusable and Single-Use Bronchoscopes.
 - Birgitte Lilholt Sørensen, Henrik Grüttner. American Journal of Environmental Protection. Vol. 7, No. 4, 2018, pp. 55-62. doi: 10.11648/j.ajep.20180704.11
- Guideline for Disinfection and Sterilization in Healthcare Facilities, 2008 Update: May 2019
 William A. Rutala, Ph.D., M.P.H., David J. Weber, M.D., M.P.H., and the

Healthcare Infection Control Practices Advisory Committee (HICPAC)

- 8) Single-use Bronchoscopes Applications in COVID-19 Pandemic Barron, Sarah MBBCh BAO; Kennedy, Marcus P. MD
- https://www.fda.gov/medical-devices/safety-communications/ flexible-bronchoscopes-and-updated-recommendationsreprocessing-fda-safety-communication

