


Galaxy System™ Setup for Increased Clearance



This document provides instructions for an alternate setup workflow that maximizes the clearance between the patient bed and the Galaxy System™ which may be used when auxiliary equipment limits the workspace.

Before You Begin

Follow the instructions until the 'Align' Step in the Instructions for Use (10000752 Rev.F, page 60) and on the Galaxy System™ Touchscreen. Begin referencing this guide on the 'Align'/'Alignment' Screen.

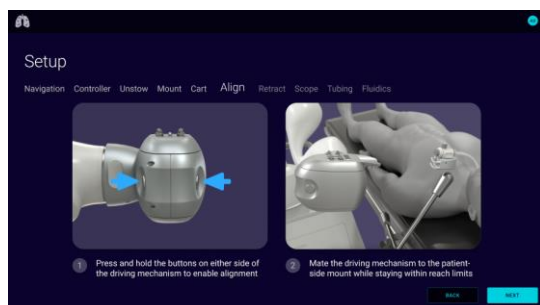


NOTE: If resistance is experienced when performing alignment, do not force the Scope Drive past the boundaries. Instead, adjust the Cart and/or Patient Side Mount such that alignment can be performed with the Scope Drive in the resistance-free zone.



CAUTION: Proper alignment between the Scope Drive and Patient Side Mount is important for smooth scope insertion and retraction as well as preventing air leakage during ventilation.

Step 1: Alignment

- 1 Unpack the Scope Guide and load onto the Scope Drive.

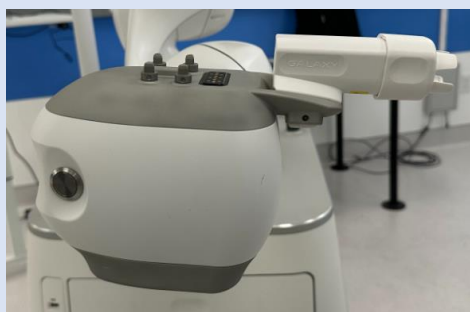

- 2 Initiate hand-guide by pressing and holding the buttons on both sides of the Scope Drive.



- 3 Hand-guide the Scope Drive towards the Bronchoscope Connector until the Bronchoscope Connector and Scope Guide connect, as indicated by an audible and tactile click.

- Hold the PSM and Bronchoscope Connector while connecting the Scope Guide and Bronchoscope Connector to prevent PSM motion.



Hold PSM & Bronchoscope Connector

✓ Correct




✗ Incorrect




Step 2: Retract the Robotic Arm


- 1 Click **NEXT** on the 'Align' Screen to advance to the 'Retract' Screen.

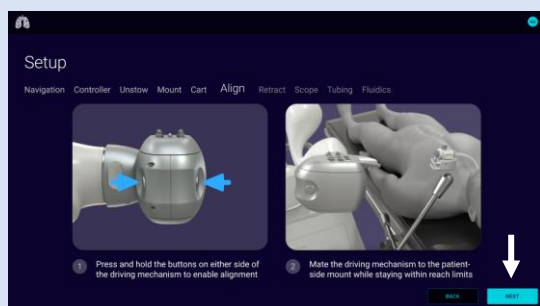


- 2 Press and hold **RETRACT** on the Touchscreen to retract the robotic arm.
 - To stop motion at any time, let go of the button. To resume, press and hold **RETRACT**.

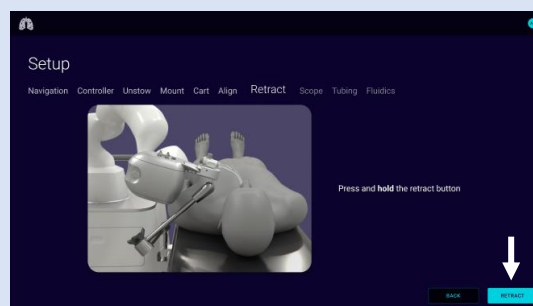

- 3 Collapse the Scope Guide.
 - Collapse along a linear trajectory to prevent kinking of the Scope Guide.

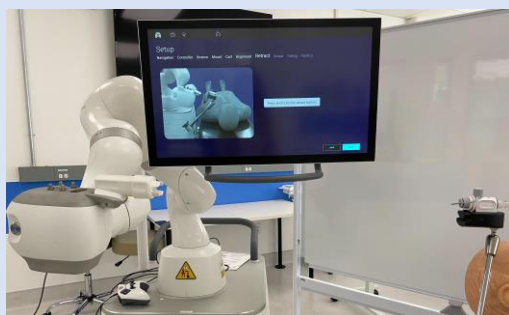

- 4 Click **NEXT** on the Touchscreen to advance to the next step.

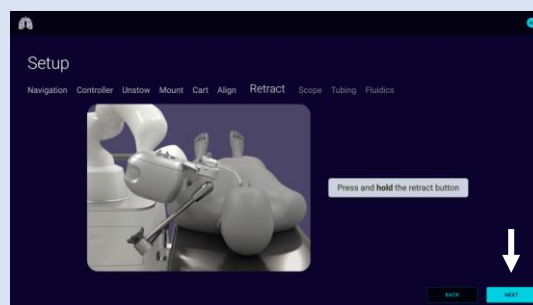

Upon Completion

Stop referencing this guide after the 'Retract' Screen.

Begin following the Instructions for Use (10000752 Rev.F, page 61), starting from 'Load the Bronchoscope' Section, Step #2 and the instructions on the Galaxy System™ Touchscreen.